

Department of Health and Human Services Health and Environmental Testing Laboratory 47 Independence Drive # 12 State House Station Augusta, Maine 04333-0012

Tel: (207) 287-2727; Fax: (207) 287-6832 TTY: 1-800-606-0215

## Measles (Rubeola) Virus Detection by Real Time RT PCR

Laboratory Submission Information Sheet

| Reporting of suspect case to Maine CDC    | •Yes. Measles is a Notifiable Condition. If you have not already done so, please report suspect or confirmed cases to the Maine CDC via the disease reporting line: 1-800-821-5821 (24hrs/day, 7 days/week.)                                                                                                                                                        |
|-------------------------------------------|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| Required To Submit<br>Laboratory Specimen | •Yes. •Collect samples for PCR within 1-3 days of rash onset.                                                                                                                                                                                                                                                                                                       |
| Required Information                      | <ul> <li>Information on requisition must include: test requested, patient name, DOB, date of collection, specimen source or type, submitter name and contact information.</li> <li>Important: all specimens must be labeled with patient name, DOB or other unique identifier, and be accompanied by a HETL requisition.</li> </ul>                                 |
| Specimen<br>Requirements                  | <ul> <li>Preferred specimen is an oropharyngeal or nasopharyngeal swab.</li> <li>ONLY synthetic-tipped swabs (Polyester, Rayon, Dacron) with plastic shafts should be used.</li> <li>Store specimens at 2-8°C until ready for shipping. Specimens should be submitted to lab as soon as possible. Use frozen cold packs to transport samples to the lab.</li> </ul> |
| Collection Instructions                   | <ul> <li>Specimens should be submitted in viral transport media or other appropriate transport media.</li> <li>Store specimens at 2-8°C until ready for shipping.</li> <li>Specimens should be submitted to lab as soon as possible. Use frozen cold packs to transport samples to the lab.</li> </ul>                                                              |
| Turn Around Time                          | • Results are generally available within 1 - 3 business days of specimen receipt                                                                                                                                                                                                                                                                                    |
| Unacceptable<br>Conditions                | • Specimens received without a form and clinical details or with insufficient material to analyze. • Specimens in poorly labeled or leaking containers • Use of wooden shaft or calcium alginate swabs • Do not freeze specimens for PCR testing                                                                                                                    |
| Results                                   | • Measles RNA Detected or Measles RNA Not Detected • Results will only be reported to the submitter listed on the requisition                                                                                                                                                                                                                                       |
| Laboratory Testing<br>Fee                 | •\$110.00 •Fee may be waived if testing is part of an outbreak investigation by Maine CDC                                                                                                                                                                                                                                                                           |
| Applicable CPT Codes                      | •87798                                                                                                                                                                                                                                                                                                                                                              |
| Additional Information                    | •Bacteriology at 207-287-1704 •Virology, Respiratory, Arbovirus, Serology, and STI at 207-287-1722 •Molecular Biology, Sequencing, and Foodborne at 207-287-5769 •Mycobacterium and Rabies at 207-287-1706                                                                                                                                                          |